


SHIPPING INSTRUCTIONS

- 1.) Chute side: Label each tube with eartag or other animal ID.
- 2.) *Use a NEW NEEDLE for each animal*
- 3.) Check total number of tubes matches number of tubes on submittal form.
- 4.) Check each tube is labeled with correct animal ID.
- 5.) Complete submittal form and enclose check for proper amount.
- 6.) Bundle up to 10 tubes with rubber band each bundle.
- 7.) Wrap 1 paper towels around each bundle.
- 8.) Place into a Ziploc style leak proof baggie.
- 9.) Pack baggies into a box with bubble wrap or other padding to keep tubes from breaking.
 - ***If sending more than 50, tubes can be sent in tube box, wrap box in cellophane and placed in ziplock baggie.***
- 10). Place biohazard symbol onto at least one baggie.

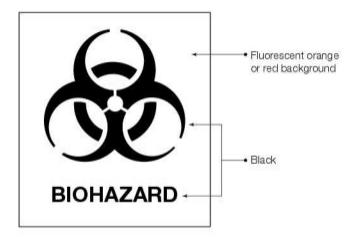

or download from this site.

http://pe.usps.com/text/DMM300/601.HTM#WP1096434

- 10.) Enclose submittal form and payment. Seal box with packaging tape.
- 11.) Address Box with label below.

NEW TEMPORARY SHIPPING LABELS*



US Mail has been better delivery thru this covid-19 situation. If you are using ups or fedex if possible please email the tracking number. Thank you!



|--|