




SHIPPING INSTRUCTIONS

- 1.) Chute side: Label each tube with eartag or other animal ID.
- 2.) *Use a NEW NEEDLE for each cow*
- 3.) Check total number of tubes matches number of tubes on submittal form.
- 4.) Check each tube is labeled with correct animal ID.
- 5.) Complete submittal form and enclose check for proper amount.
- 6.) Bundle up to 10 tubes with rubber band each bundle.
- 7.) Wrap 2 paper towels around each bundle.
- 8.) Place into a Ziploc style leak proof baggie.
- 9.) Pack baggies into a box with bubble wrap or other padding to keep tubes from breaking.
- 10.) Place biohazard symbol onto at least one baggie.

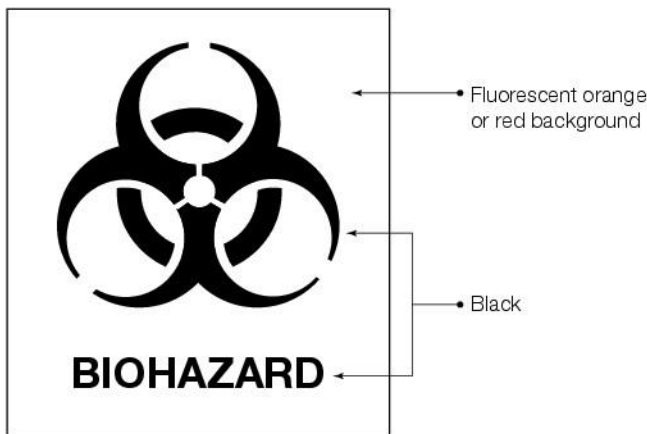

or download from this site.


<http://pe.usps.com/text/DMM300/601.HTM#WP1096434>

Enclose submittal form and payment. Seal box with packaging tape.

****If you are sending 50 or more tubes – Place tubes in the original box. Wrap Saran Wrap around box 2 times one direction then two times the opposite. Place in gallon size baggie with paper towel. Then place in shipping box.**

This will hold tubes in place and prevent leaking/breaking and is easier than bundles of 10.

11.) Address Box with label below.

<hr/> <hr/> <hr/>
 Eagle Talon Enterprises, L.L.C. P.O. Box 924 Laramie, Wy 82073

UPS/FedEx ↓

<hr/> <hr/> <hr/>
 Eagle Talon Enterprises, L.L.C. 514 E. Grand Ave. #164 Laramie, WY 82070

12.) If packaged correctly: ADD this label to outside of box.

“EXEMPT ANIMAL SPECIMEN”